Digi Connect[®] WS

Hardware Reference Manual

Digi Connect WS Hardware Reference Manual

(Part number 90001459)

Revision	Date	Description
A	December 2015	Initial release of the Digi Connect WS Hardware Reference Manual
B	February 2016	Applied minor corrections and updated the Electromagnetic immunity table

Disclaimers

Information in this document is subject to change without notice and does not represent a commitment on the part of Digi International. Digi provides this document “as is,” without warranty of any kind, expressed or implied, including, but not limited to, the implied warranties of fitness or merchantability for a particular purpose. Digi may make improvements and/or changes in this manual or in the product(s) and/or the program(s) described in this manual at any time.

Trademarks and copyright

Digi, Digi International, and the Digi logo are trademarks or registered trademarks in the United States and other countries worldwide. All other trademarks mentioned in this document are the property of their respective owners.

© 2016 Digi International. All rights reserved.

Warranty

View the product warranties online: <http://www.digi.com/howtobuy/terms>

Customer support

Digi offers multiple technical support plans and service packages to help our customers get the most out of their Digi product. For information on Technical Support plans and pricing, please contact us at 952.912.3456 or visit us at www.digi.com/support.

Telephone (8:00 am — 5:00 pm CST):

US & Canada: 877.912.3444

Worldwide: +1 952.912.3456

Online: www.digi.com/support/eservice

Mail:

Digi International

11001 Bren Road East

Minnetonka, MN 55343

US

Contents

Digi Connect WS Hardware Reference Manual..... 2

About the Digi Connect WS

Overview of the Digi Connect WS	14
Device description	14
Intended use	15
Connect WS server models	15
Essential performance	15
Target audience	16
Safety information	16
Warnings, cautions, and notes	16
Important safety information	17

Connect WS Hardware

Connect WS System	21
Connect WS models	21
Powering the Connect WS	21
Mounting the Connect WS	21
Connect WS device components	22
Connect WS LED functions and descriptions	23
RJ-45 pinouts for the Connect WS	24
Connect WS specifications	24
Safety and regulatory standards	25

Using the Connect WS

Powering the device on and off	26
Cleaning and inspection	27
Connect WS connectivity options	27
Connect WS configuration	27
Access the web UI using the device IP address	28
Access the web UI using the Digi Device Discovery Utility	28

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Connect WS settings and descriptions	30
Connect WS home page	30
Device information	31
Refreshing the home page	31

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Overview of the Digi Connect WS

Device description

The Digi Connect WS, an electronic data transmitting device, is intended to provide electronic transfer of device data without controlling or altering the function or parameters of any device to which it is connected. As part of a network system, the Connect WS is intended to receive and transmit data to/from devices to which it is connected. It does not incorporate a therapeutic or diagnostic function.

The Connect WS is intended to be used with a data acquisition system to connect to RS-232 serial devices and send data from these serial devices to the data acquisition system without adjusting or manipulating the format. Connect WS complies with IEC 60601-1 3rd edition, and can be used in environments requiring this level of testing, including being placed within the patient environment when used as part of a medical electrical (ME) system.

The Connect WS provides serial over IP connectivity for the environment that connects RS-232 devices to an information system network. There are three models of the Connect WS: the Connect WS 1, the Connect WS 4, and the Connect WS 8, which provide one, four, and eight RS-232 serial ports, respectively. The Connect WS includes a 10/100 Mbps network interface card, and an 802.11a/b/g/n compatible Wi-Fi card.

The Connect WS is intended solely to provide an interface between devices with serial ports and network connected computers running a monitoring application to collect data from those devices.

To transmit data, the serial ports on the Connect WS are connected physically to serial ports on medical devices. Computers connect through the network to the Ethernet port or Wi-Fi card of the Connect WS and communicate via serial data communication over TCP or UDP with the Connect WS. The Connect WS, when configured, automatically transmits serial data to and from connected serial devices with network-connected computers using TCP or UDP sockets. Without changing or manipulating the data, the Connect WS allows for bi-directional data communication between serial connected medical devices and network connected computers (through either Ethernet or Wi-Fi).

There is no electrical path for current to earth ground, ensuring no electrical shock when making physical contact with the Connect WS Terminal Server.

The Connect WS does not provide alarm or safety notification capabilities and is not to be used in the critical path between any device and any alarm or safety notification system.

Intended use

The Connect WS, an electronic data transmitting device, is intended to provide electronic transfer of data from devices with serial connections to information systems via IP networks. The Connect WS does not control or alter the functions or parameters of any device to which it is connected. It does not change or manipulate transmitted data or its display in any way. It does not incorporate a therapeutic or diagnostic function, and is not intended to be used with active patient monitoring.

The Connect WS is intended for use as an ITE device for data transmission only.

Note Any use of this product beyond the scope of the above description is at the customer's sole risk and responsibility.

Connect WS server models

This guide pertains to all Connect WS server models:

Base model	Regional model variants	Description
DC-WS-1	DC-WS-1-???*	Digi Connect WS 1 (1-port)
DC-WS-4	DC-WS-4-???*	Digi Connect WS 1 (4-port)
DC-WS-8	DC-WS-8-???*	Digi Connect WS 1 (8-port)

*Regional variants may have optional characters (denoted with -???) that indicate different accessory options.

Power and cabling requirements

Use an appropriate power cable meeting national standards to connect to a standard outlet. The appropriate requirements are listed per region:

- EU/International: VDE Mark, conforming to IEC 60083, IEC 60227, or IEC 60320, with C7 to the appropriate national mains connector, 2 x 0.75 mm².
- USA/Canada:
 - UR Mark, conforming to UL 62, UL 817, or CSA-C22.2, with C7 to 1-15P, 5-15P, or NEMA locking connector, 18 AWG.
 - If Power over Ethernet is used, the device shall use a CAT5 or higher cable that has been inspected and approved by Facility IT and which provides connectivity on all pins.

Included accessories

- 1 Ethernet dust cover
- 4 VESA mount screws

Essential performance

The essential performance of the Connect WS is to not compromise the data coming from the attached device (excluding issues which might cause delayed delivery of data or data not being delivered), and to not compromise the operation of the connected device.

The expected service life of the Connect WS is 7 years.

Target audience

This guide is intended for those who are responsible for setting up and configuring Digi devices, and who are trained to operate the equipment connected to the Connect WS to ensure that the product is safe and fully functional.

Installation requirements

Make sure the Connect WS installation is performed by IT personnel with at least one year of experience operating network equipment within the intended environment.

Any additional required configuration or troubleshooting for the Connect WS is performed remotely with the third party software and equipment and follows specific procedures provided by the Systems Integrator.

Personnel connecting and configuring devices to the Connect WS should have knowledge of how these devices work and the proper procedures for configuring their serial ports, as defined in the documentation provided with the equipment.

The personnel responsible for configuring and verifying proper communication between an attached device and the data acquisition system that is monitoring this data should have knowledge of the correct configuration methods and procedures, as defined in the materials provided with the data acquisition system.

The System Integrator is responsible to ensure compliance with any country-specific regulations that pertain when installing or implementing a solution that uses this product.

CAUTION! Do not modify the Connect WS terminal server or cables, or mount it in any way other than instructed in this guide. Any modification to the hardware can prevent proper functioning and cause damage to the connected device.

WARNING! When deploying the Connect WS in an environment subject to IEC 60601-1, refer to the requirements applicable to medical equipment systems in the standard.

Safety information

Warnings, cautions, and notes

Where necessary, the documentation highlights any applicable warnings, cautions, or notes to help eliminate or avoid the risk of injury to individuals or damage to equipment. The following lists the levels of information:

CAUTION! Indicates a hazard with a low level of risk, which, if not avoided, could result in minor to moderate injury or severe damage or malfunctioning of the equipment.

WARNING! Indicates a hazard with a medium to high level of risk which, if not avoided, can result in death or serious injury.

Note This indicates helpful information to assist the user, and to avoid the risk of damage to equipment described in this guide, and/or any other equipment or goods.

Important safety information

WARNING! In case of smoke disconnect device and discontinue use. To safely terminate operation of the Connect WS, disconnect the power cable and the Ethernet cable.

WARNING! To avoid contact with an electrical current:

- Do not touch device if immersed in liquid.
- Do not touch device and discontinue use if enclosure is broken.
- Never install electrical wiring during an electrical storm.
- Never install an Ethernet connection in wet locations unless that connector is specifically designed for wet locations.
- Use caution when installing or modifying lines.
- Use a screwdriver and other tools with insulated handles.
- Wear safety glasses or goggles.
- Do not place Ethernet wiring or connections in any conduit, outlet or junction box containing electrical wiring.
- As a best practice, mount the device server on a wall or place it on a tabletop away from high traffic areas. Keep cables out of reach and off the floor.
- Do not place power cable in walkways or anywhere it may receive heavy wear or be crushed, or where it may be a trip hazard.
- Only use screws included in the packaging.
- Do not connect, remove, or modify protectors or grounding wires; you must have a qualified service provider perform these tasks.
- Do not touch uninsulated Ethernet wiring if lightning is likely.
- External wiring: Any external communications wiring installed must be constructed to all relevant electrical codes. In the United States, follow the National Electrical Code, Article 800. Contact a licensed electrician for details.

WARNING! Verify correct installation and configuration:

- Position the Connect WS so that the power connector and power switch face up. Do not position the Connect WS within 15 centimeters (6 inches) of the equipment above it. Position the device so that it is easy to remove and re-insert the power mains cable and the Ethernet cable.
- Do not use a multiple socket-outlet or an extension cord to connect this device.
- Device provides a maximum of 1.5 KV of isolation power protection for Power over Ethernet. Installer must ensure connected devices provide additional isolation if required.
- The serial ports of the Connect WS shall only be connected to 60601 Class IIa or IIb or 60950 equipment.
- If using the Connect WS in a ME environment, connect only device items that have been specified as part of the ME system or specified as being compatible with the ME system.
- Do not use near any device that emits electromagnetic waves, such as large motors or electrosurgical equipment, or the data may be corrupted.
- Reciprocal interference may be caused during specific investigations or treatments. Do not use this equipment during diagnosis or treatment unless you have verified that no reciprocal interference is caused.
- Do not rely on this equipment as part of an alarm or alert system.
- Disconnect all devices from the device server before using diagnostic mode.

- Install the device in secure networks with Internet firewalls.
- Wi-Fi connections may be subject to unexpected interference and loss of signal.
- Check network configuration and serial connection mapping before using device for data transfer.

WARNING! Connecting to IT networks that include other equipment could result in previously unidentified risks to patients, operators, or third parties. The responsible organization shall identify, analyze, evaluate, and control these risks.

WARNING! Changes to the IT network could introduce new risks that require additional analysis. Changes to the IT-network include:

- Network configuration changes
- Connecting additional items
- Disconnecting items
- Updating equipment including update of firmware to the Connect WS
- Upgrading equipment

CAUTION! Do not share or write down usernames or passwords.

WARNING! Do not use this device in areas with high magnetic fields.

WARNING! Do not connect the Ethernet cable to a serial port. Damage may occur to the serial pins, causing the device to malfunction.

CAUTION! This equipment should be connected only to other equipment compliant with IEC 60601 or IEC 60950.

Recycling and disposal

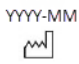
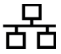
	Improper disposal of IT equipment can have a negative impact on health and the environment. Dispose of equipment and accessories at an appropriate facility to enable recovery and recycling.
	Recycle packaging and guides according to your local recycling regulations.

Environmental information

	Operating temperature	0° C to +40° C (32° F to 104° F)
	Storage temperature	-20° C to 60° C (-4° F to 140° F)
	Relative humidity	10% to 90% (non-condensing)
	Operating atmospheric pressure	800 hPa to 1060 hPa
	Storage atmospheric pressure	500 hPa to 1060 hPa

Symbol definitions

	Atmospheric pressure limitation
	Humidity limitation
	Temperature limitation
DIM	Device carton shipping dimensions
ETH	Ethernet MAC address
GW	Gross weight
NW	Net weight
REF	Catalog number
SN	Serial number
WLAN	Wi-Fi MAC address

	Mark of conformity
	ALERT: This product employs frequencies that are not harmonized throughout the EC. Potential restrictions on the use of this apparatus may exist in one or more member states.
	Operating instructions
	Date of manufacture
	Manufacturer
	Ethernet port and status LED
	Serial port(s) and status LED(s)
	Power status LED
	Wi-Fi connection status LED
	Find ME LED
	Alternating current
	Caution
	Warning
	Package recycling information
	WEEE Device was put on the market after 13 August 2005. Conforms to the Directive 2003/108/EC on waste electrical and electronic equipment (WEEE) of the European Parliament.

Connect WS Hardware

Connect WS System

Connect WS models

The Connect WS terminal server has three models, consisting of one, four, or eight RS-232 serial ports with 8p/8c RJ-45 connectors. This device also includes a single RJ-45 802.3 10/100 Ethernet port with Power over Ethernet capability. Moreover, the Connect WS provides 802.11 a/b/g/n Wi-Fi capability with embedded main and diversity antennas.

Powering the Connect WS

The Connect WS is powered via Power-over-Ethernet (PoE) or an AC mains power cord. If both PoE and a power cord are plugged into the device, it will access power via the AC power supply. In this case, a minimum current is drawn from the PoE supply to maintain the power signature of the Connect WS. The PoE in the Connect WS is responsible for keeping the power signature continuous.

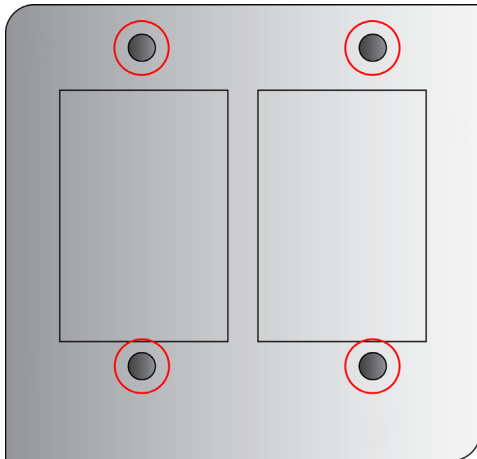
During power transitions from AC/DC to PoE and vice versa, the device may or may not go through a reset; therefore, seamless power transition is not guaranteed in either direction. After you have connected the power supply and powered the device on, the Power LED will be solid green. In case of system failure, the Connect WS is designed to maintain its settings through the use of its internal non-volatile storage.

Mounting the Connect WS

The Connect WS features a VESA 75 mounting pattern located on the back of the enclosure. This gives you the option to fix mount the device on a wall or cart/stand using a 75 x 75 mm VESA adapter. The following image displays the VESA mounting pattern.

The VESA adapter and tools needed for installation are not included. Commercially available VESA mounts and mounting tools may be used by personnel familiar with their use. Maximum torque for the VESA screws should not exceed 2.5 N.m (22 in-lbf).


Connect WS VESA mounting pattern


Connect WS device components

The following images show the location of the Connect WS hardware components along with associated LED indicator descriptions.

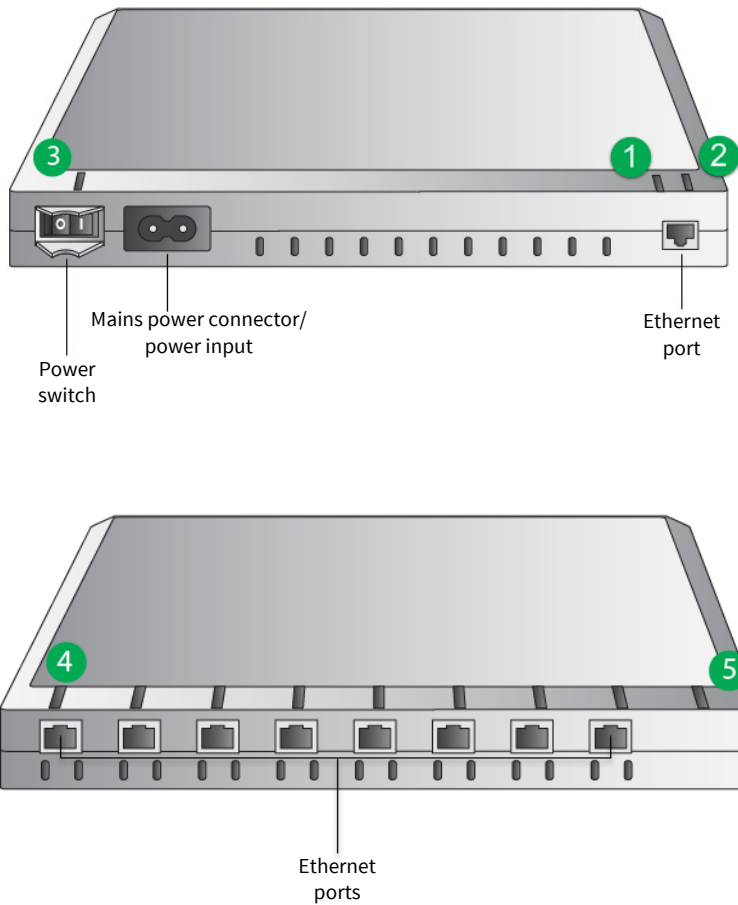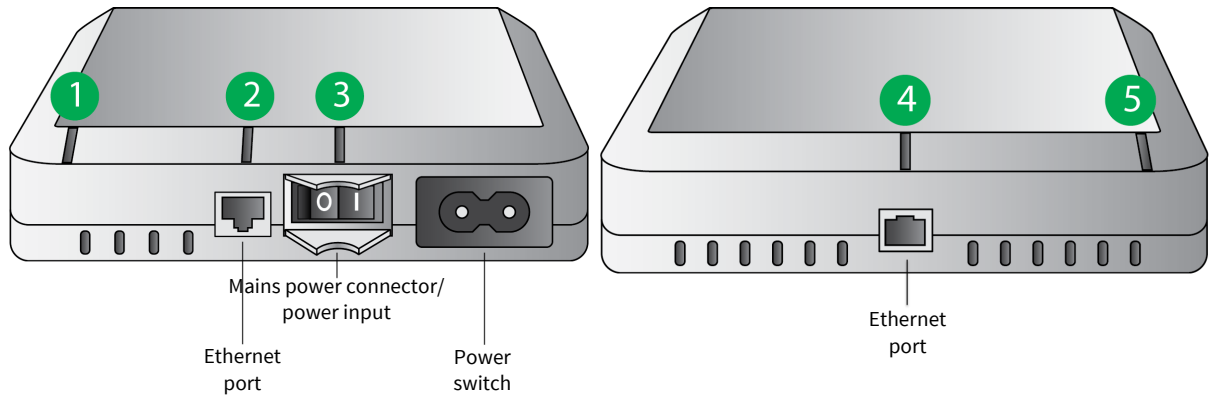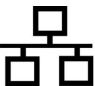
Connect WS enclosure callouts/LED locations (4-port or 8-port model)

Connect WS enclosure callouts/LED locations (1 port model)

Connect WS LED functions and descriptions

	LED/button	Color and light pattern	Description
	Wi-Fi LED (1)	Solid green	The device is connected to an access point.
		Flashing green	The device is attempting to connect to an access point.
		Solid amber	The device is scanning for a network.
		Off	Wi-Fi is disabled.
	Ethernet LED (2)	Solid green	There is a connection to the device, but there is no traffic on the line.
		Flashing green	There is traffic on the connection.
		Off	There is no connection.
	Power (3)	Solid green	The device is on and receiving power.
		Off	The device is off and not receiving power.
IOIOI	Serial LEDs (4)	Solid green	TCP connection is established to the serial port, but the TCP connection is not in use.
		Off	TCP connection is not in use.
	Find ME LED (5)	Flashing amber	Find ME LED is on, which allows finding the device among a group of devices.
		Off	Find ME LED is turned off.

RJ-45 pinouts for the Connect WS

Pin assignments for the RJ-45 connector on the Connect WS are as follows:

RJ-45 pin number	EIA-232 signal
1	DSR
2	RTS
3	GND
4	TxD
5	RxD
6	SG (GND)
7	CTS
8	DTR

Connect WS specifications

Specification		Value
Environmental	Operating temperature	0° C to +40° C
	Storage temperature	-20° C to +60° C (4° F to 140° F)
	Operating atmospheric pressure	800 to 1060 hPa
	Relative humidity	10% to 90% (non-condensing)
	Operating altitude	2000 meters
Power requirements	External	100-240 VAC / 50-60 Hz WS1: 0.1A RMS Max WS4-8: 0.25A RMS Max
	Power-over-Ethernet (PoE)	802.3af compliant
Wi-Fi		802.11a/b/g/n, 2.4 GHz and 5 GHz capable with built-in antennas (main and diversity)
Ethernet	Data rate	10/100 Mbps
RS232 Serial communications		1, 4, or 8 RJ45 RS232 serial ports
		Supports standard baud rates up to 230400 bps
Dimensions	Connect WS 1	135 mm x 115 mm x 30 mm
	Connect WS 4/8	200 mm x 157 mm x 36 mm

Safety and regulatory standards

Category	Standard
Medical electrical equipment — General requirements for basic safety and essential performance	EN 60601-1: 2005 + corr.1 (2006) + corr.2 (2007) + A1 :2013
Medical electrical equipment — Part 1-2: General requirements for basic safety and essential performance Collateral standard: Electromagnetic compatibility — requirements and tests	IEC 60601-1-2 Ed3 : 2007
Medical electrical equipment — Part 1-6: General requirements for basic safety and essential performance Collateral standard: Usability	IEC 60601-1-6: 2006 + A1:2013
Medical device software lifecycle processes	EN 62304:2006
Medical device usability	EN 62366: 2008
EMC/EMI	FCC 47 CFR Part 15 part B and C ICES-003
Radio	EN 301 489-1 v1.9.2
	EN 301 489-17 v2.1.1.1
	EN 300 328 v1.8.1
	EN 62311: 2008
	RSS-210
OET65	ANSI/IEEE C95.1-1992
Wi-Fi	Wi-Fi Alliance Certified

Note The safety and regulatory information in this section is provided for information purposes only, and is believed to be accurate at the time of publication of this manual. Please refer to the Declaration of Conformity for current safety and regulatory information. The above categories reference the common names of standards and do not necessarily reflect intended uses. The Connect WS does not change or manipulate transmitted data or its display in any way. It does not incorporate a therapeutic or diagnostic function, and is not intended to be used with active patient monitoring. The Connect WS is intended for use as an ITE device for data transmission only.

Using the Connect WS

Powering the device on and off

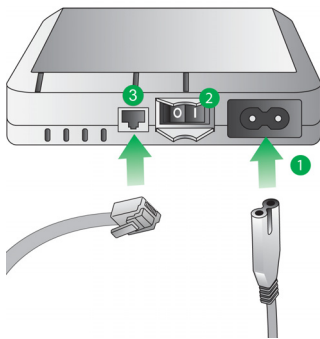
To power on the device:

- 1 Connect a C7 power cord to the mains power connector/power input (1) on the Digi Connect WS terminal server or you can connect an Ethernet cable (3) to a PoE-enabled Ethernet switch.
- 2 Turn on the power switch (2). The Power LED (2) illuminates.

To power off the device:

- 1 Turn off the power switch (2). The Power LED (2) is no longer illuminated.
- 2 Disconnect the power cord from the mains power connector (1) and disconnect the Ethernet cable from the Ethernet port (3).

Power-on sequencing for the Connect WS (1)

WARNING! To disconnect the Connect WS from power, you must remove both the mains power cord and the Ethernet cable.

Note Before cleaning or moving the device, turn off the power and disconnect the device from the mains power connector and the Ethernet cable.

Note The Connect WS also supports 802.3af Power over Ethernet (PoE). You can use both mains power and PoE at the same time. No additional configuration is required to enable PoE on the device.

The LEDs will illuminate as booting occurs, then shut off as the unit finishes booting. The Find ME LED blinks amber initially, signifying that the device is attempting to complete the start-up boot process. After the boot process is complete, the Find ME LED shuts off, and you can now configure the device through a web user interface (UI).

Wi-Fi is disabled by default, so first-time configuration of the Connect WS terminal server must be done via an Ethernet connection. You can make subsequent configurations or modifications to the device through the web UI. This can either be done via Ethernet or Wi-Fi, once Wi-Fi capabilities have been enabled.

WARNING! Do not connect the Ethernet cable into a serial port. Damage may occur to the serial pins, causing malfunction of the device.

Cleaning and inspection

WARNING! Clean and inspect the device periodically:

- Periodically inspect power cable and replace power cable in case it shows wear.
- Clean device regularly.
- Do not sterilize the device server. (To clean: gently wipe the case with a mild detergent or 70% isopropyl alcohol.)
- Disconnect the mains power cord and the Ethernet cable from the Connect WS before cleaning the Connect WS or inspecting the power cord or other cables for wear.

CAUTION! Verify correct operation before each use and periodically thereafter.

- Do not use the device server to activate, deactivate, control, or adjust any medical device.
- Do not use data from the device server for diagnosis without validating the integrity of the entire information collection system.

WARNING! This device contains no user serviceable parts. No modification of this equipment is allowed. The manufacturer cannot take responsibility for any damage caused by opening the device.

Connect WS connectivity options

Initial configuration of the Connect WS terminal server is done via Ethernet. The device must be powered on, and connected to a local area network (LAN) through the use of a desktop or laptop that is equipped with an Ethernet port. Once you have connected the device via an Ethernet connection, the Digi Device Discovery Utility, or your own tool can be used to discover your device on your local area network. Once the initial setup and configuration is complete, you can make subsequent configurations or modifications to the Connect WS terminal server via:

- 1 Ethernet
- 2 Wi-Fi (Wi-Fi capabilities must be enabled through the device's web UI)

Obtaining and working with the Device Discovery Tool will be discussed in later sections of this manual.

Connect WS configuration

To configure your device, you must know how to access the device's web interface. You can access the web UI by entering the device's IP address (the IP address is obtained via DHCP), into the URL address bar of your browser. If user and password authentication are not enabled, your browser will automatically redirect to your device's

configuration settings Home page. If you don't know your device's IP address, you can access the web UI of your device in a similar way by using the Digi Device Discovery Utility.

The web UI, which is available via a local area network connection to the device, provides an easy way to configure your device settings and perform administrative tasks.

The following sections demonstrate how to access the Connect WS web interface by using an IP address, as well as the Digi Device Discovery Utility. Device information displayed within the web UI varies by model.

Access the web UI using the device IP address

Basic HTTP authentication is enabled in the device. You will need to know the administrative username and password to access the device's configuration settings page.

- 1 Open your preferred web browser.
- 2 In the URL address bar, enter the IP address of your device.
- 3 You will be prompted to enter the username and password to access the device. If this is the first time you are accessing the device's web UI, enter **admin** as the default username, and **admin** as the default password.
- 4 The device's configuration settings Home page is displayed.

Note If you are trying to access your device's web UI for the first time, and the default administrative username and password do not work, contact your system administrator; the username and password may have been changed.

Access the web UI using the Digi Device Discovery Utility

The Digi Device Discovery Utility locates Digi devices on a network. It allows you to open the web interface for discovered devices, configure network settings, and reboot the device. It uses a Digi International-proprietary protocol, Advanced Digi Discovery Protocol (ADDP), to discover devices on a network, and displays the discovered devices in a list.

The Digi Device Discovery Utility is available for download on the Digi support site. To download the Digi Device Discovery Utility, proceed with the following steps:

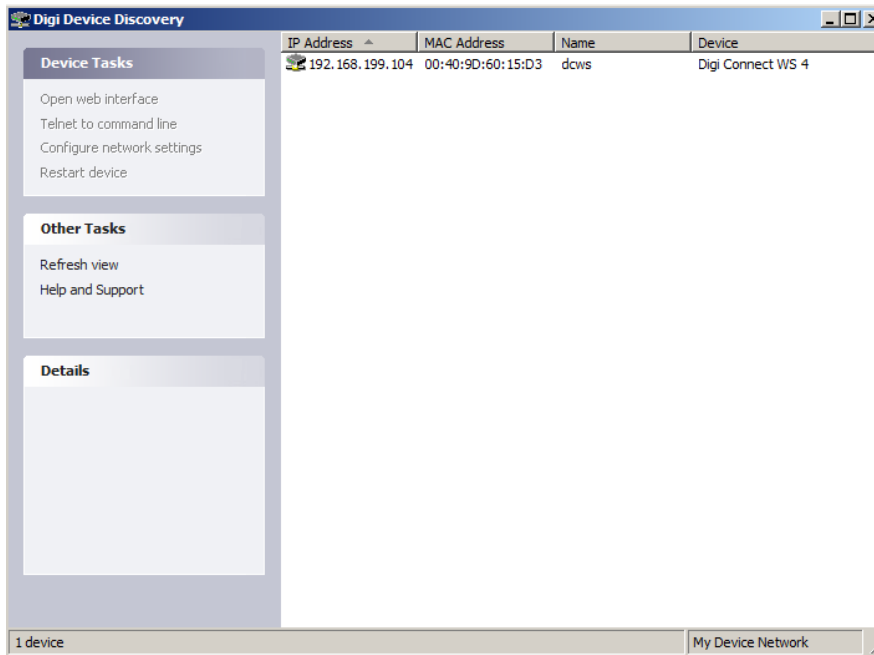
- 1 Open your web browser, and navigate to www.digi.com/support.
- 2 Under **Select Your Product for Support**, select **Device Discovery** from the product list.
- 3 Select **Diagnostics, Utilities and MIBs** in the navigation bar located on the left side of the page.
- 4 Under **Operating System Specific Utilities**, select your appropriate operating system from the drop-down menu.
- 5 Select **Device Discovery Utility for Windows-Standalone Version**. The standalone version runs the utility immediately after the download is complete.

Discover your device

Once you have downloaded and installed the Digi Device Discovery Utility:

- 1 Open the program by double-clicking the program icon on your desktop, or by navigating to **Start > Programs > Digi > Digi Device Discovery > Digi Device Discovery**.
- 2 The Digi Device Discovery Utility is displayed. Locate your device among the list of devices.
- 3 Double-click your device name or select it from the list and click **Open Web Interface** in the Device Tasks list.

Digi Device Discovery Utility

Understanding the Connect WS web interface

Connect WS settings and descriptions

There are many options available on the Connect WS device home page to help manage and monitor your device. This chapter covers the different features and settings available within the web UI, along with brief descriptions regarding the functionality of each setting.

Connect WS home page

When you open the Connect WS web interface, the device home page is displayed. Information listed on this page will vary depending on the product and its supported features. The following is an example of the Connect WS Home page.

Connect WS home page

The screenshot displays the Connect WS web interface home page. On the left is a navigation menu with sections for Configuration, Administration, and Reboot. The main content area is titled 'Device Information' and is divided into two sections: 'Current System Status' and 'Network Connectivity Status'. The 'Current System Status' section lists various system parameters, and the 'Network Connectivity Status' section shows a list of checked status items.

Device Information	
Current System Status	
Model:	Digi Connect WS 8 PROTOTYPE
Product ID:	0x804f
Firmware Version:	3.2.9.5 cws release gw-3.2.9.5 08/22/2014 15:07:00 CDT
Device ID:	00000000-00000000-00409dff-ff60120f
Host Name:	dcws
Date and Time:	Thu May 21 18:07:19 UTC 2015
Up Time:	6 minutes 31 seconds
Service Hours:	192
Serial Logging:	Enabled
Ethernet MAC Address:	00:40:9d:60:12:0f
Ethernet IP Address:	10.35.1.107
Wi-Fi MAC Address:	00:40:9d:60:12:10
DNS Servers:	10.10.8.62, 10.10.8.64, 8.8.8.8
Time Servers:	time.etherios.com, 0.idigi.pool.ntp.org, 1.idigi.pool.ntp.org, 2.idigi.pool.ntp.org
CPU Utilization:	34%
Total Memory:	126756 KB
Used Memory:	33468 KB
Free Memory:	93288 KB
Network Connectivity Status ■	
<input checked="" type="checkbox"/>	Network interface detected.
<input checked="" type="checkbox"/>	Connected to local network.
<input checked="" type="checkbox"/>	IP address assigned.
<input checked="" type="checkbox"/>	Prepared to contact Device Cloud server.
<input checked="" type="checkbox"/>	Contacted Device Cloud server.
<input checked="" type="checkbox"/>	Ready for Device Cloud communication.
<input type="button" value="Refresh"/>	

Device information

The Device Information section summarizes current system parameters and network connectivity status. The Network Connectivity Status LED along with the information displayed below it, indicates the readiness of the Connect WS to communicate in a network.

Refreshing the home page

By clicking the **Refresh** button, you are refreshing the page. Refreshing a page is necessary when doing certain things like viewing system time or network connectivity status. Utilizing the refresh button is important because certain features are not automatically updated when the state of the device changes. The Refresh button also updates the device status information.

Applying and saving changes

The web interface runs locally on the Connect WS terminal server, which means that the interface always maintains and displays the latest settings of the device. On each screen, the Apply button saves any configuration changes made to the device.

Canceling changes

To cancel changes made to a configuration settings page, click the Refresh or Reload icon on your web browser. Pressing the refresh icon on your browser causes the browser to reload the page. Any changes made since the last time the Apply button was clicked are reset back to their original value.

Restoring the Connect WS to factory defaults

You can reset your Connect WS device to factory defaults as needed during the configuration process. You can do this through the device's web UI, or by pressing and holding the **Reset** button on the device for about 2 seconds.

Any network configuration settings will have to be restored and saved after the reset operation is complete.

Connect WS configuration settings

On the left side of the Home page there is a menu of choices for configuration and administrative tasks. The configuration settings allow you to configure settings for various features.

Ethernet network settings

The Ethernet network settings page displays the current IP address and DHCP settings for Ethernet network communications. You can change the IP address by either obtaining a new one through DHCP, or by entering a static IP address.

Current IP parameters

This section displays the current IP address assignment for your Connect WS terminal server. If DHCP is enabled, the IP, subnet mask, and default gateway address will be assigned automatically.

Stored IP configuration

This section displays how the IP address for the Connect WS is obtained. You can obtain an IP address for your device two different ways; either automatically using DHCP, or manually by entering a static IP address, subnet mask, and default gateway.

By default, the Connect WS terminal server uses a DHCP server to obtain its IP address information. A DHCP server automatically provides an IP address, subnet mask, default gateway, and Domain Name System (DNS) server for your device.

If DHCP is disabled, an IP address is assigned through AutoIP, and all configuration settings need to be done manually. In the absence of a DHCP server, a static IP address needs to be assigned. You can do this by accessing the device through the Digi Device Discovery Utility, and changing the network settings through the web UI.

IP address

The IP address is a 4-part ID assigned to network devices. An IP address is like a telephone number for a computer where each number is between 0 and 255. Other network devices can talk to your Connect WS device using this ID.

Subnet mask

The subnet mask is combined with the IP address to determine which network your device is a part of. A common Subnet Mask looks something like 255.255.255.0.

Default gateway

The default gateway is the IP address of the computer that allows your device to access other networks such as the Internet.

Domain name service configuration

The Domain Name Service Configuration section shows the two domain name system servers that are to be used as static servers when dynamic mechanisms do not supply enough DNS servers.

Wireless network settings

Similar to the Ethernet Network settings page, the wireless network settings page displays the current IP address and DHCP settings for wireless network communications.

Wi-Fi configuration wizard

The Wi-Fi configuration wizard configures the security parameters of the Wi-Fi interface. This wizard gives a Connect WS device the wireless parameters needed to further configure wireless settings and operations.

Note Before running the wizard, confirm that Wi-Fi is enabled by checking “Enable this network interface.”

SSID

The SSID should be set to the name of your wireless access point. If your device is being configured in the environment where it will be used, the SSID should be available among the list of SSIDs in step one of the Wi-Fi configuration wizard.

If your device is being configured before it is deployed in the environment where it will be used, then it is likely that the SSID may need to be entered manually in the “Network SSID” field.

Step two of the Wi-Fi configuration wizard prompts you to enter the security mode of your device and its associated parameters.

Security mode

The security mode option sets the type of network security for the wireless network.

Security parameters in the Wi-Fi configuration wizard

Device Configuration and Management
Digi Connect WS 4

Wireless Network Configuration Wizard

Step 2: Select Security Mode and enter associated parameters:

Security Mode:

EAP Type:

Username:

Password:

Confirm Password:

Click "Finish" to continue or "Cancel" to end the wizard and return to the Wireless Network Configuration page.

Note WPA/WPA2 Enterprise is the recommended setting for creating a secure network.

If “WPA/WPA2 Enterprise” security is selected, there are two additional subcategories of wireless security that are supported.

EAP type

The EAP Type setting allows for more enhanced security measures for your device. The EAP Type drop-down list provides two options. LEAP, which is a much simpler security feature, only uses a username and password for additional security, while PEAP-MSCHAPv2, a more complex security feature, allows for even more security options.

Username and password

The username and password fields give you the option of setting a unique username and password for your device. Utilize the username and password settings to enhance the wireless security.

Completing the Wi-Fi configuration wizard

To complete the wireless wizard configuration settings:

- 1 Enter the SSID, or choose an SSID from the list of IDs, if supplied. The SSID of your device must be known in order to proceed.
- 2 Select the type of network security for the wireless network.
- 3 Depending on your network security type, fill in the additional parameters for enhanced wireless security.
- 4 Click **Finish** at the bottom of the page when you are done filling in all fields. Messages are displayed while the security parameters are being processed, and when the wizard is complete.

Note If the Wi-Fi connection does not establish (the LED keeps returning to Amber) confirm the correct settings to be used with the IT staff. If in doubt test connectivity with an Access Point with a known configuration.

Serial port settings

The Serial Port configuration page establishes parameters for serial ports to send and receive data. The Serial Port configuration page includes the currently selected port profile for the serial port, and detailed configuration settings for the serial port. The following are brief descriptions of the basic fields within the serial port configuration settings.

Note To configure your serial ports, click one of the ports you would like to configure (outlined in red).

Serial port settings page for the Connect WS 4

Serial Port Configuration			
Current Settings			
Port	Description	TCP Port	Current Settings
Port 1		2101	19200 8N1
Port 2		2102	19200 8N1
Port 3		2103	19200 8N1
Port 4		2104	19200 8N1

Serial Port 1 Configuration Settings for the Connect WS 4

Serial Port 1 Configuration			
Basic Serial Settings			
Description:	<input type="text"/>	(optional)	
Baud Rate:	<input type="text" value="19200"/>	Data Bits:	<input type="text" value="8"/>
Parity:	<input type="text" value="none"/>	Stop Bits:	<input type="text" value="1"/>
Flow Control:	<input type="text" value="none"/>		
TCP Keep-Alive Settings			
<input checked="" type="checkbox"/>	Enable TCP Keep-Alive		
Idle Timeout:	<input type="text" value="0"/> hrs	<input type="text" value="5"/> mins	<input type="text" value="0"/> secs (10 secs - 9 hrs)
Probe Interval:	<input type="text" value="10"/> secs (10-75)		
Probe Count:	<input type="text" value="5"/> probes (5-30)		
<input type="button" value="Apply"/>			

Description

The Description field specifies an optional character string for the port which can be used to identify the device connected to the port.

Basic serial settings

Basic serial settings include Baud Rate, Data Bits, Parity, Stop Bits, and Flow Control. The basic serial settings must match the serial settings of the connected device. If you do not know these settings, consult the documentation that came with your serial device. These serial settings may be documented as 9600 8N1, which means that the device is using a baud rate of 9600 bits per second, 8 data bits, no parity, and 1 stop bit.

Network services

The Network Services configuration page is a set of common network services that are available for your device, and the port on which the service is running. You can enable or disable these network services, and the TCP port on which the network services listen can be changed from the default, with some exceptions. This means that the use of network services on your device can be restricted to the services your device needs. To improve device security, disable non-secure services. You can disable services for security reasons so that your device is only running the services it needs. The following are configurable services for the Connect WS.

Connect WS network service configuration

Network Service Configuration

Current Settings

Secure Shell (SSH)
Allows users secure access to log in to the Digi device and access the command-line interface.

Enable Secure Shell Server (SSH) TCP Port:

WEB Server (HTTP)
Web pages for configuration.

Enable Web Server (HTTP) TCP Port:

Secure WEB Server (HTTPS)
Web pages encryption to improve the security of web data transfers.

Enable Secure Web Server (HTTPS) TCP Port:

Device Discovery Service (ADDP)
Discovery of Digi devices on a network using Digi Device Discovery application.

Device Discovery Service (ADDP) Mode: **Read-Only:** Allows discovery of devices.
Read-Write: (default) Allows discovery and network configuration.
Off: Disables service entirely.

Secure Shell Server (SSH)

The Secure Shell Server is not accessible to users, it is only for manufacturer use. It can be left disabled in normal operation.

Hypertext Transfer Protocol (HTTP)

Hypertext Transfer Protocol over Secure Socket Layer (HTTPS)

This device provides web pages for configuration that can be secured by requiring a user login.

Advanced Digi Discovery Protocol (ADDP)

The Advanced Digi Discovery Protocol (ADDP) runs on any operating system capable of sending multicast IP packets on a network. ADDP allows the system to identify all ADDP-enabled Digi devices attached to a network by sending out a multicast packet. The devices respond to the multicast packet and identifies themselves to the client sending the multicast. ADDP communicates with the TCP/IP stack using UDP. The TCP/IP stack should be able to receive multicast packets and transmit datagrams on a network.

Access to ADDP service can be enabled or disabled, but the network port number for ADDP cannot be changed from its default.

Simple Network Management Protocol (SNMP)

Simple Network Management Protocol (SNMP) is a protocol for managing and monitoring network devices. SNMP architecture enables a network administrator to manage nodes-servers, workstations, routers, switches, hubs on an IP network, manage network performance, find and solve network problems, and plan for network growth.

Connect WS administrative settings

This section discusses the administration tasks that need to be performed on your Connect WS device periodically, such as file management, changing the password used for logging onto the device, backing up and restoring device configurations, updating firmware and Boot/POST code, restoring the device configuration to factory defaults, and rebooting the device.

Backup/restore

Once your Connect WS device is configured, it is good practice to back up your configuration settings in case problems occur later, firmware is upgraded, or hardware is added. If you need to configure multiple devices, you can use the backup/restore feature. Back up the first device's configuration settings to a file, then load the file onto the other devices.

Firmware update

You can update the firmware code for the Connect WS from a file on a PC. The recommended method is to download the firmware to a local hard drive.

Reboot

Changes to some device settings require saving the changes and rebooting the Digi device. To reboot the device:

- 1 From the web interface menu, select **Administration > Reboot**.
- 2 On the **Reboot** page, click the **Reboot** button. Wait approximately 1 minute for the reboot to complete .

Find Me LED

The Find Me LED on the Connect WS helps you to find a specific Digi device server among a group of devices.

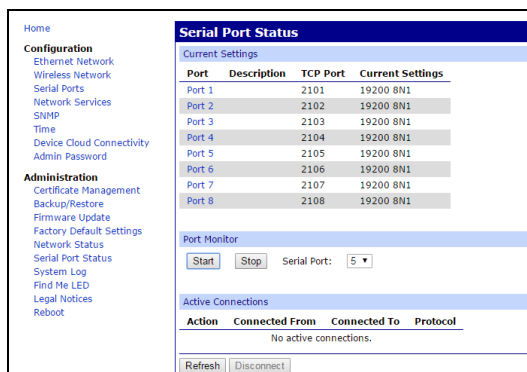
- **Activate:** Clicking this button causes the Find Me locator LED to blink.
- **Stop:** Clicking this button causes the Find Me LED to stop blinking.

Port logging

The Connect WS supports a port logging feature that is intended for technical validation or troubleshooting and debugging purposes only. Do not enable this feature when the device is in normal operational use. Port logging is automatically disabled during a power cycle of the device.


To enable this feature on one port for troubleshooting purposes:

- 1 On the **Administration** menu, click **Serial Port Status**. The **Serial Port Status** window appears.
- 2 In the **Port Monitor** section, select the port number from the **Serial Port** drop-down menu.
- 3 Click **Start**.

To view port logging data:

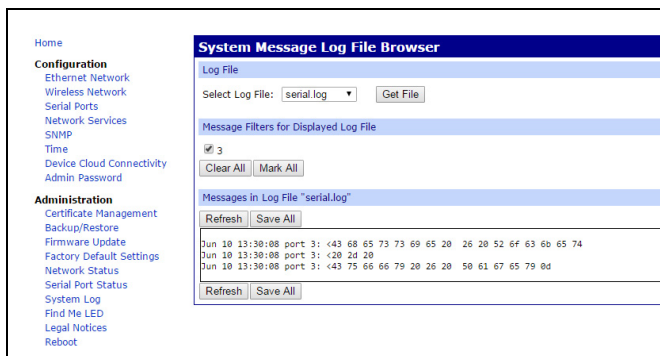- 1 On the Administration menu, click **System Log**. The **System Message Log File Browser** window appears.
- 2 From the **Select Log File** drop-down menu, select **serial.log**.
- 3 Click **Get File**.

CAUTION! You must power cycle the device after using this feature for troubleshooting and before placing the device in operation.

Additional considerations:

- There are no operator controls on the device.
- Other than these instructions for placement and initial setup and configuration, there are no additional requirements for operator access or positioning of the device.
- No additional setup is required from the operator.
- Any additional required configuration or troubleshooting is done remotely following the specific procedures that the Systems Integrator supplies for use of this product with their software.
- If you need to troubleshoot the device while it is in use, you must refer to the instructions from the Systems Integrator whose software is being used. Always contact the Systems Integrator when you are unsure how to proceed when using the device.
- For product RMA or questions concerning specific instructions in this manual, contact Digi Technical Support. However, Digi cannot give guidance on the correct method of use or suitability of use of this product with any third party equipment or software.

Electromagnetic Guidance and manufacturer's declaration

Electromagnetic effects

Connect WS is an electronic device that both emits and absorbs intentional and unintentional electromagnetic radiation. It is designed and tested to comply with EN/IEC 60601-2. Improper installation not following this user guide may cause this device to interfere with other electronic devices, including, but not limited to, Medical Electrical (ME) equipment.

Consideration must also be taken regarding the potential presence of portable or mobile RF communications equipment that may also impact nearby equipment.

The customer or user needs to make sure that recommendations of other equipment manufacturers are followed especially concerning accessories, power supplies and cables for any device that is in proximity to the Connect WS or that is connected to it, otherwise increased emission or reduced immunity may occur.

Electromagnetic emissions

Connect WS is designed and tested to comply and operate within the limits specified below.

Emissions test	Compliance	Electromagnetic environment guidance
RF emissions CISPR 11	Group 1	The Connect WS is a very low power device and generates very low levels of Radiated Spurious Emissions. The potential for unwanted interference with nearby electronic devices is very low.
RF emissions CISPR 11	Class A	The Connect WS is suitable for use in all establishments other than domestic and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Harmonic emissions IEC 61000-3-2	Class A	The Connect WS is a very low power device and generates very low levels of Radiated Spurious Emissions. The potential for unwanted interference with nearby electronic devices is very low.
Voltage fluctuations/flicker emissions IEC 61000-3-3	Complies	None.

Electromagnetic immunity

Connect WS is intended for use in electromagnetic environments within the limits specified below. The customer or user of Connect WS should assure that it is used in such an environment.

Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment guidance
Electrostatic discharge (ESD) IEC 61000-4-2	±6 kV contact	±6 kV contact	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30%.
	±8 kV air	±8 kV air	
Electrical fast transient/burst IEC 61000-4-4	±2 kV for power supply lines	±2 kV for power supply lines	A typical commercial or hospital environment mains power source is suitable.
	±1 kV for input/output Lines	±1 kV for input/output Lines	
Surge IEC 61000-4-5	±1 kV line(s) to line(s)	±1 kV line(s) to line(s)	A typical commercial or hospital environment mains power source is suitable.
	±2 kV line(s) to earth	±2 kV line(s) to earth	

Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment guidance
Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11	<5% U_T (>95% dip in U_T) for 0.5 cycle	>95% voltage dips in U_T for 0.5 cycles	A typical commercial or hospital environment mains power source is suitable. If the user of the Connect WS requires continued operation during power mains interruptions, it is recommended that the Connect WS be powered from an uninterruptible power supply.
	40% U_T (60% dip in U_T) for 5 cycles	60% voltage dips in U_T for 5 cycles	
	70% U_T (30% dip in U_T) for 25 cycles	30% voltage dips in U_T for 25 cycles	
	<5% U_T (>95% dip in U_T) for 5 sec	>95% voltage interruption in U_T for 5 seconds	
Power frequency (50/60 Hz) magnetic field IEC 61000-4-8	3 A/m	3 A/m	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.

U_T is the AC mains voltage prior to application of the test level.

Electromagnetic immunity

Connect WS is intended for use in electromagnetic environments within the limits specified below. The customer or user of Connect WS should assure that it is used in such an environment.

Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment guidance
Conducted RF IEC 61000-4-6	3 Vrms 150 kHz to 80 MHz	3 V	See next section for recommended separation distance Over the frequency range 150 kHz to 80 MHz, field strengths should be less than 3 V/m.
Radiated RF IEC 61000-4-3	3 V/m 80 MHz to 2.5 GHz	3 V/m	See next section for recommended separation distance

Note Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey, should be less than the compliance level in each frequency range. Interference may occur in the vicinity of equipment marked with this symbol:

Recommended separation distances

Use the following recommended separation distances between RF communications equipment and Connect WS.

P (Manufacturer's Rated Maximum Output Power of transmitter in Watts)	d (Recommended Separation distance in Meters according to frequency of transmitter)		
	150 kHz to < 80 MHz	80 MHz to < 800 MHz	800 MHz to 2.5 GHz
	$d = 1.16 \sqrt{P}$	$d = 1.16 \sqrt{P}$	$d = 2.33\sqrt{P}$
0.01	0.11	0.11	0.23
0.1	0.36	0.36	0.73
1	1.16	1.16	2.33
10	3.68	3.68	7.37
100	11.66	11.66	23.33

Notes:

This table is an aid to quickly determine the recommended distance for typical frequencies and power levels. The provided formula can be used to calculate the distance for other power levels, where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer. In case of multiple transmitters at different frequencies the minimum recommended distance is the largest distance calculated from the different methods.

These are generic guidelines for near field RF objects and may not apply in all situations. Actual field strength may be impacted by reflection and absorption of any object, surface, and so on.

Field strength for far field radiation, such as cellular communication, cordless devices, mobile radios, broadcast radio and TV signals cannot be predicted with this method. Only an electronic site survey of the Electric Field Strength can assess the potential risk of exceeding the maximum field strength. If field strength can vary or may exceed the recommended maximum values, the proper operation of the device should be monitored or other preventive measures, such as relocation, reorientation or other measures may be necessary.

Any RF communications equipment, including any connected cables, should be kept at a distance from the Connect WS, including any connected cables, that is larger than the recommended separation distance calculated from the equation applicable to the power and frequency of the transmitter.

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Connect WS-hjemmeside

The screenshot displays the web interface for a Connect WS device. On the left is a navigation menu with sections for Configuration and Administration. The main content area is titled 'Device Information' and shows 'Current System Status' with various system parameters. Below this is a 'Network Connectivity Status' section with a green indicator and a list of checked status items.

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Network Connectivity Status	
<input checked="" type="checkbox"/>	Network interface detected.
<input checked="" type="checkbox"/>	Connected to local network.
<input checked="" type="checkbox"/>	IP address assigned.
<input checked="" type="checkbox"/>	Prepared to contact Device Cloud server.
<input checked="" type="checkbox"/>	Contacted Device Cloud server.
<input checked="" type="checkbox"/>	Ready for Device Cloud communication.

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Connect WS-Startseite

The screenshot displays the web interface of a Connect WS device. On the left is a navigation menu with sections for Configuration, Administration, and Reboot. The main content area is titled 'Device Information' and is divided into two sections: 'Current System Status' and 'Network Connectivity Status'.

Current System Status:

- Model: Digi Connect WS 8 PROTOTYPE
- Product ID: 0x804f
- Firmware Version: 3.2.9.5 cws release gw-3.2.9.5 08/22/2014 15:07:00 CDT
- Device ID: 00000000-00000000-00409dff-ff60120f
- Host Name: dcws
- Date and Time: Thu May 21 18:07:19 UTC 2015
- Up Time: 6 minutes 31 seconds
- Service Hours: 192
- Serial Logging: Enabled
- Ethernet MAC Address: 00:40:9d:60:12:0f
- Ethernet IP Address: 10.35.1.107
- Wi-Fi MAC Address: 00:40:9d:60:12:10
- DNS Servers: 10.10.8.62, 10.10.8.64, 8.8.8.8
- Time Servers: time.etherios.com, 0.idigl.pool.ntp.org, 1.idigl.pool.ntp.org, 2.idigl.pool.ntp.org
- CPU Utilization: 34%
- Total Memory: 126756 KB
- Used Memory: 33468 KB
- Free Memory: 93288 KB

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A 'Refresh' button is located at the bottom of the Network Connectivity Status section.

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Luokka	Standardi
Medical electrical equipment — General requirements for basic safety and essential performance	EN 60601-1: 2005 + corr.1 (2006) + corr.2 (2007) + A1: 2013
Medical electrical equipment — Part 1-2: General requirements for basic safety and essential performance Collateral standard: Electromagnetic compatibility — requirements and tests	IEC 60601-1-2 Ed3: 2007
Medical electrical equipment — Part 1-6: General requirements for basic safety and essential performance Collateral standard: Usability	IEC 60601-1-6: 2006 + A1: 2013
Medical device software lifecycle processes	EN 62304: 2006
Medical device usability	EN 62366: 2008
EMC/EMI	FCC 47 CFR Part 15 part B and C ICES-003
Radio	EN 301 489-1 v1.9.2
	EN 301 489-17 v2.1.1
	EN 300 328 v1.8.1
	EN 62311: 2008
	RSS-210
OET65	ANSI/IEEE C95.1-1992
Wi-Fi	Wi-Fi Alliance Certified

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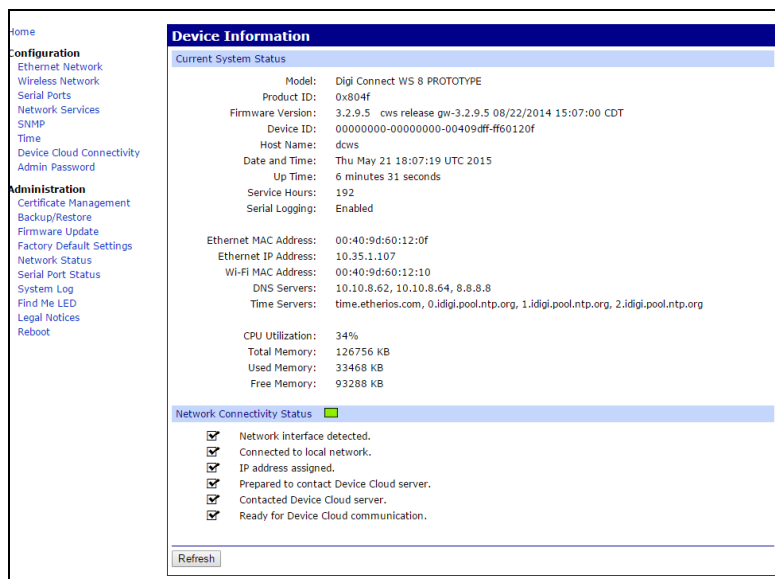
The screenshot displays the 'Device Information' page of the Connect WS web interface. On the left, there is a navigation menu with categories like 'Configuration' and 'Administration'. The main content area is titled 'Device Information' and shows 'Current System Status' with various system parameters. Below this, there is a 'Network Connectivity Status' section with a green indicator and a list of checked status items.

Current System Status	
Model:	Digi Connect WS 8 PROTOTYPE
Product ID:	0x804f
Firmware Version:	3.2.9.5 cws release gw-3.2.9.5 08/22/2014 15:07:00 CDT
Device ID:	00000000-00000000-00409dff-ff60120f
Host Name:	dcws
Date and Time:	Thu May 21 18:07:19 UTC 2015
Up Time:	6 minutes 31 seconds
Service Hours:	192
Serial Logging:	Enabled
Ethernet MAC Address:	00:40:9d:60:12:0f
Ethernet IP Address:	10.35.1.107
Wi-Fi MAC Address:	00:40:9d:60:12:10
DNS Servers:	10.10.8.62, 10.10.8.64, 8.8.8.8
Time Servers:	time.etherios.com, 0.idigi.pool.ntp.org, 1.idigi.pool.ntp.org, 2.idigi.pool.ntp.org
CPU Utilization:	34%
Total Memory:	126756 KB
Used Memory:	33468 KB
Free Memory:	93288 KB

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The screenshot displays the web interface of a Connect WS device. On the left is a navigation menu with categories like Configuration, Administration, and Reboot. The main content area is titled 'Device Information' and shows 'Current System Status' with various system parameters. Below this, there is a 'Network Connectivity Status' section with a green indicator and a list of checked status items.

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Home page di Connect WS

The screenshot displays the web interface for a Connect WS device. On the left is a navigation menu with sections for Configuration, Administration, and Reboot. The main content area is titled 'Device Information' and is divided into two sections: 'Current System Status' and 'Network Connectivity Status'.

Configuration Menu:

- Ethernet Network
- Wireless Network
- Serial Ports
- Network Services
- SNMP
- Time
- Device Cloud Connectivity
- Admin Password

Administration Menu:

- Certificate Management
- Backup/Restore
- Firmware Update
- Factory Default Settings
- Network Status
- Serial Port Status
- System Log
- Find Me LED
- Legal Notices
- Reboot

Device Information - Current System Status:

Model:	Digi Connect WS 8 PROTOTYPE
Product ID:	0x804f
Firmware Version:	3.2.9.5 cws release gw-3.2.9.5 08/22/2014 15:07:00 CDT
Device ID:	00000000-00000000-00409dff-ff60120f
Host Name:	dcws
Date and Time:	Thu May 21 18:07:19 UTC 2015
Up Time:	6 minutes 31 seconds
Service Hours:	192
Serial Logging:	Enabled

Device Information - Network Connectivity Status:

Ethernet MAC Address:	00:40:9d:60:12:0f
Ethernet IP Address:	10.35.1.107
Wi-Fi MAC Address:	00:40:9d:60:12:10
DNS Servers:	10.10.8.62, 10.10.8.64, 8.8.8.8
Time Servers:	time.etherios.com, 0.idigi.pool.ntp.org, 1.idigi.pool.ntp.org, 2.idigi.pool.ntp.org

Network Connectivity Status: ■

- Network interface detected.
- Connected to local network.
- IP address assigned.
- Prepared to contact Device Cloud server.
- Contacted Device Cloud server.
- Ready for Device Cloud communication.

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Connect WS-homepagina

The screenshot displays the Connect WS web interface. On the left is a navigation menu with categories: Configuration (Ethernet Network, Wireless Network, Serial Ports, Network Services, SNMP, Time, Device Cloud Connectivity, Admin Password), Administration (Certificate Management, Backup/Restore, Firmware Update, Factory Default Settings, Network Status, Serial Port Status, System Log, Find Me LED, Legal Notices, Reboot), and a 'home' link. The main content area is titled 'Device Information' and is divided into two sections. The top section, 'Current System Status', lists: Model: Digi Connect WS 8 PROTOTYPE, Product ID: 0x804f, Firmware Version: 3.2.9.5 cws release gw-3.2.9.5 08/22/2014 15:07:00 CDT, Device ID: 00000000-00000000-00409dff-ff60120f, Host Name: dcws, Date and Time: Thu May 21 18:07:19 UTC 2015, Up Time: 6 minutes 31 seconds, Service Hours: 192, and Serial Logging: Enabled. The bottom section, 'Network Connectivity Status', shows a green indicator and a list of checked items: Network interface detected, Connected to local network, IP address assigned, Prepared to contact Device Cloud server, Contacted Device Cloud server, and Ready for Device Cloud communication. A 'Refresh' button is located at the bottom of this section.

Device Information	
Current System Status	
Model:	Digi Connect WS 8 PROTOTYPE
Product ID:	0x804f
Firmware Version:	3.2.9.5 cws release gw-3.2.9.5 08/22/2014 15:07:00 CDT
Device ID:	00000000-00000000-00409dff-ff60120f
Host Name:	dcws
Date and Time:	Thu May 21 18:07:19 UTC 2015
Up Time:	6 minutes 31 seconds
Service Hours:	192
Serial Logging:	Enabled
Network Connectivity Status ■	
<input checked="" type="checkbox"/>	Network interface detected.
<input checked="" type="checkbox"/>	Connected to local network.
<input checked="" type="checkbox"/>	IP address assigned.
<input checked="" type="checkbox"/>	Prepared to contact Device Cloud server.
<input checked="" type="checkbox"/>	Contacted Device Cloud server.
<input checked="" type="checkbox"/>	Ready for Device Cloud communication.

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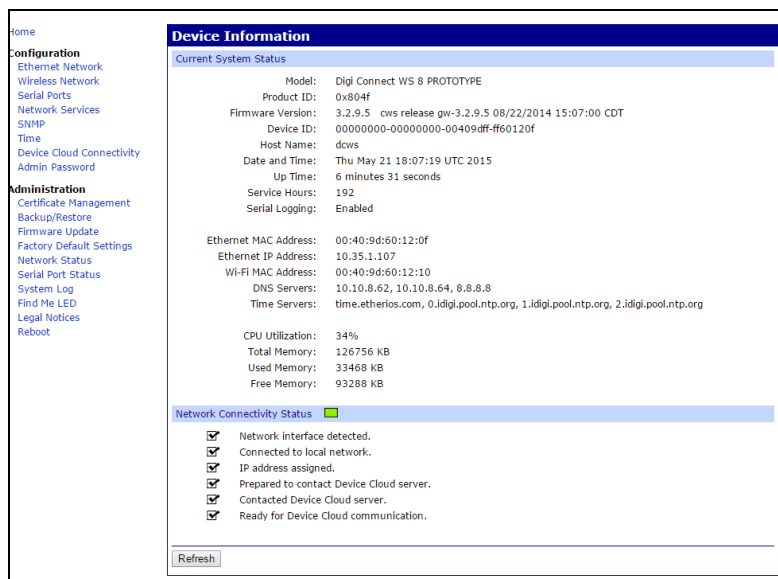
Connect WS – hjemmeside

The screenshot displays the web interface for a Connect WS device. On the left is a navigation menu with sections for Configuration and Administration. The main content area is titled 'Device Information' and shows 'Current System Status' with various system parameters. Below this, the 'Network Connectivity Status' is shown as green, with a list of checked items indicating network readiness.

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Network Connectivity Status	
[Green indicator]	
<input checked="" type="checkbox"/>	Network interface detected.
<input checked="" type="checkbox"/>	Connected to local network.
<input checked="" type="checkbox"/>	IP address assigned.
<input checked="" type="checkbox"/>	Prepared to contact Device Cloud server.
<input checked="" type="checkbox"/>	Contacted Device Cloud server.
<input checked="" type="checkbox"/>	Ready for Device Cloud communication.

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Connect WS-startsidan

The screenshot displays the web interface for a Connect WS device. On the left is a navigation menu with categories like Configuration, Administration, and Reboot. The main content area is titled 'Device Information' and shows 'Current System Status' with various system parameters. Below this, there is a 'Network Connectivity Status' section with a green indicator and a list of checked status items.

Device Information	
Current System Status	
Model:	Digi Connect WS 8 PROTOTYPE
Product ID:	0x804f
Firmware Version:	3.2.9.5 cws release gw-3.2.9.5 08/22/2014 15:07:00 CDT
Device ID:	00000000-00000000-00409dff-ff60120f
Host Name:	dcws
Date and Time:	Thu May 21 18:07:19 UTC 2015
Up Time:	6 minutes 31 seconds
Service Hours:	192
Serial Logging:	Enabled
Ethernet MAC Address:	00:40:9d:60:12:0f
Ethernet IP Address:	10.35.1.107
Wi-Fi MAC Address:	00:40:9d:60:12:10
DNS Servers:	10.10.8.62, 10.10.8.64, 8.8.8.8
Time Servers:	time.etherios.com, 0.idigi.pool.ntp.org, 1.idigi.pool.ntp.org, 2.idigi.pool.ntp.org
CPU Utilization:	34%
Total Memory:	126756 KB
Used Memory:	33468 KB
Free Memory:	93288 KB

Network Connectivity Status	
[Green Indicator]	
<input checked="" type="checkbox"/>	Network interface detected.
<input checked="" type="checkbox"/>	Connected to local network.
<input checked="" type="checkbox"/>	IP address assigned.
<input checked="" type="checkbox"/>	Prepared to contact Device Cloud server.
<input checked="" type="checkbox"/>	Contacted Device Cloud server.
<input checked="" type="checkbox"/>	Ready for Device Cloud communication.

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